



EUROREPAR

WORKSHOP CONSUMABLE

# Disposable protection masks

>95%

**Bacterial  
Filter  
Efficiency**

**ADAPTED  
OFFER**  
for  
professionals  
in contact with  
customers



**LES + PRODUIT**

- 3 filter protection (Polypropylene, filter)
- Protection mask for professionals in contact with customers.
- Single use disposable masks.
- Good Quality worth cost.

**REF. : 1677219780**  
**50 MASKS PER BOX**



## Precautions for use

- Before putting the mask, wash your hands with water and soap or with an hydro alcoholic solution.
- When wearing a mask, do not touch it or displace it.
- Once you touch a used mask, wash your hands with water and soap or with an hydro alcoholic solution.
- If you need to drink or eat something, change your mask.
- When a mask is moistened, replace it with a new one, do not re-use, single-use masks.
- To remove the mask :
  - Remove it from the ears, do not touch the front of the mask,
  - Throw the mask in a closed trash once,
  - Wash your hands with water and soap or with an hydro alcoholic solution,
- Do not use if you are allergic to non-woven materials.
- Do not use if the packaging is damaged or opened.
- Disposable mask: must be destroyed after use, no washing and no re-use.
- Keep in a clean and ventilated area.
- Must not be used in a medical environment.

## TECHNICAL FEATURE

- Type I mask.
- **Bacterial Filter Efficiency (BFE) > 95%.**
- **Ventilating Resistance = 26 Pa/cm<sup>2</sup>** (standard : not > 49).
- **Ethylene oxyde residue < 10 µg/g.**

## OPERATING MODE

- 1) Take one mask from box,
- 2) Loosen the belt, put the mask on your face, and belts behind ears, make sure the nose bridge bar is on top,
- 3) Press the nose bridge bar with fingers from middle to both sides,
- 4) Fit the mask to your face.



## REGULATIONS

- Conformity declaration and test report are attached to the product.
- CE certification
- Standard EN 14683.



[www.eurorepar.com](http://www.eurorepar.com)